

Wind/Rain Shield Instructions

1. Attach the #2 canopy frame to the mobility scooter.
2. Attach the U shaped bar with the black fittings as shown on Fig 2.
3. Attach the frame to the front of the canopy frame as shown on Fig 3.
4. Attach the bottom of the frame to the bumper bar or bracket using the Hose clamps provided.
5. Adjust height of the rods using sliders on figure 4. Fit cross bar Fig2.
6. Attach clear side and rear shields with the velcro tabs folding over the tube from the inside to the outside.

Note. Do not overtighten fittings

